# Lansinoh®

## CONTACT NIPPLE SHIELDS

#### GB INSTRUCTIONS FOR USE

Lansinoh® Contact Nipple Shields can be effective short term tools in supporting you breastfeed where one or more of the following occurs:

- your baby’s suck or attachment is affected by tongue/lip tie, prematurity, illness or small babies
- your nipples are short, inverted or flat
- your nipples are sore and/or you are engorged (swollen)
- Overactive let-down

Before using a nipple shield, please speak to a breastfeeding expert who may be able to help you with some of the above concerns.

Using a nipple shield may encourage your baby to learn to latch whilst waiting to see a tongue-tie specialist and if your baby was premature or sick and has been fed with a bottle, then a nipple shield may help your baby to latch onto your breast.

You may not need to use a nipple shield before your milk supply has been established, as hand expression of your colostrum will provide all the nourishment your baby needs whilst you are both learning positioning and attachment.

For inverted, short or flat nipples you may also find using Lansinoh® LatchAssist™ helpful.

For sore nipples you could ask a breastfeeding expert to observe a feed to ensure that your baby’s latch is correct.

For engorged breasts you may find that hand massage, hand expressing and/or using heat therapy, such as Lansinoh® Thera®Pearl® 3-in-1 Breast Therapy packs, prior to feeding will help your baby latch on. Sometimes reverse pressure softening can help or use Lansinoh® LatchAssist™.

Where you have a forceful let-down (milk release) you could feed your baby half-lying-half-sitting; this may help your breastmilk to flow slower.

Nipple Shields allow you to continue breastfeeding while these short term latch-on issues are addressed & resolved. Nipple Shields should be used only until the situation is corrected, so it is important to address the underlying cause of breastfeeding issues via a breastfeeding expert. Should you encounter a reduction in the amount of breastmilk you produce you may need to use a breast pump to help increase your levels.

**WARNING! Nipple shields should be used under the direction of a breastfeeding expert. They should not be relied upon for long term use. Nipple shields should not be used in the first week of birth unless specifically advised by a breastfeeding expert, as their use may affect milk supply. Make sure that your nipple shield is the proper fit to ensure good milk transfer and avoid nipple damage. Keep out of reach of children when not in use.**

#### CLEANING & CARE

Follow these guidelines unless specifically advised otherwise by your healthcare provider.

- Before first use, sanitize the nipple shields by boiling in water for 10 minutes.
- Before and after each use: Wash the nipple shields in hot, soapy water.
- Regularly wash the protective case in hot, soapy water and dry thoroughly before storing the nipple shields.
- Store in the protective case with the nipple shields facing up.

#### DIRECTIONS FOR USE

- Wash your hands and the nipple shields in hot, soapy water and rinse thoroughly before applying to the breast.

- For best results, soak your nipple shield in clean, hot water for 1-2 minutes prior to breastfeeding. This makes them more pliable for a better fit on the breast. Drain the water & wait to cool slightly before applying.

- Apply a small amount of Lansinoh® HPA® Lanolin only to the rim of the nipple shield (not the nipple portion). This will help keep the shield in place leaving both hands free to position and attach your baby.
- Partially turn the nipple shield inside out by bringing the long edges toward the shield’s nipple.
- The cut out area of the nipple shield should be placed where the baby’s nose will be when feeding. Your baby’s nose should not be touching your breast or the nipple shield.

- Bring the shield to your breast, lining your nipple up with the shield’s nipple opening.

- Push the rim of the shield toward your breast turning it right side out and enclosing as much of your nipple as possible in the nipple opening.

#### WEANING FROM NIPPLE SHIELDS

Nipple shields are intended for short term use. Once your concern has been overcome, your baby may be able to breastfeed directly from the breast. You may find that your baby only needs a few more nursing sessions before you stop using a nipple shield, you may also find that your baby will want to use a shield every other feed or one feed in three. You will need to gauge whether you baby wants to feed with or without the shield. But you can also try the following:

- Hand express or pump until let-down occurs – this will elongate your nipple – have your baby ready to latch as soon as you can.
- Offer without the shield at each feed but if your baby is getting upset then have a shield ready to slip on.
- During a feed you can also slip the shield off quickly to see if your baby will continue to feed without it.

If your nipples are sore and cracked, use Lansinoh® HPA® Lanolin to soothe and protect – there is no need to remove before breastfeeding. Remember you can also get support from a breastfeeding expert through one of the national helpline or in person from your health professional or at a local breastfeeding group.

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